# **Lookin'Body120**

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Height  Feet  Inch

Gender  Male  Female

Date of Birth  Mo.  Day  Yr. \* For children under 18, input date of birth to determine the exact age.

Age

**Optional** Show

Start the InBody Test immediately after registration. Register

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**Register New** Print Form

**Required**

Name	David K. Lee						
ID	130805-1	<input checked="" type="checkbox"/> * Auto-assign ID?	<input checked="" type="radio"/> Yes <input type="radio"/> No				
<small>* Lowercase alphabets and numbers only (4-14 characters)</small>							
Height	180	cm					
Gender	<input checked="" type="radio"/> Male <input type="radio"/> Female						
Date of Birth	3	Mo.	10	Day	1987	Yr.	<small>* For children under 18, input date of birth to calculate the exact age.</small>
Age	26						

**Optional** Show

Start the InBody Test immediately after registration. **Register**

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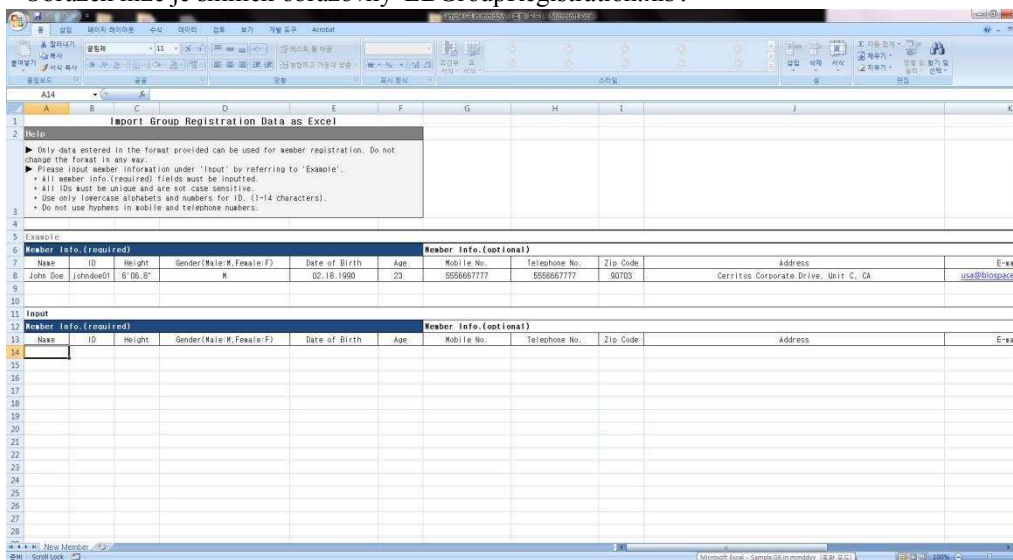
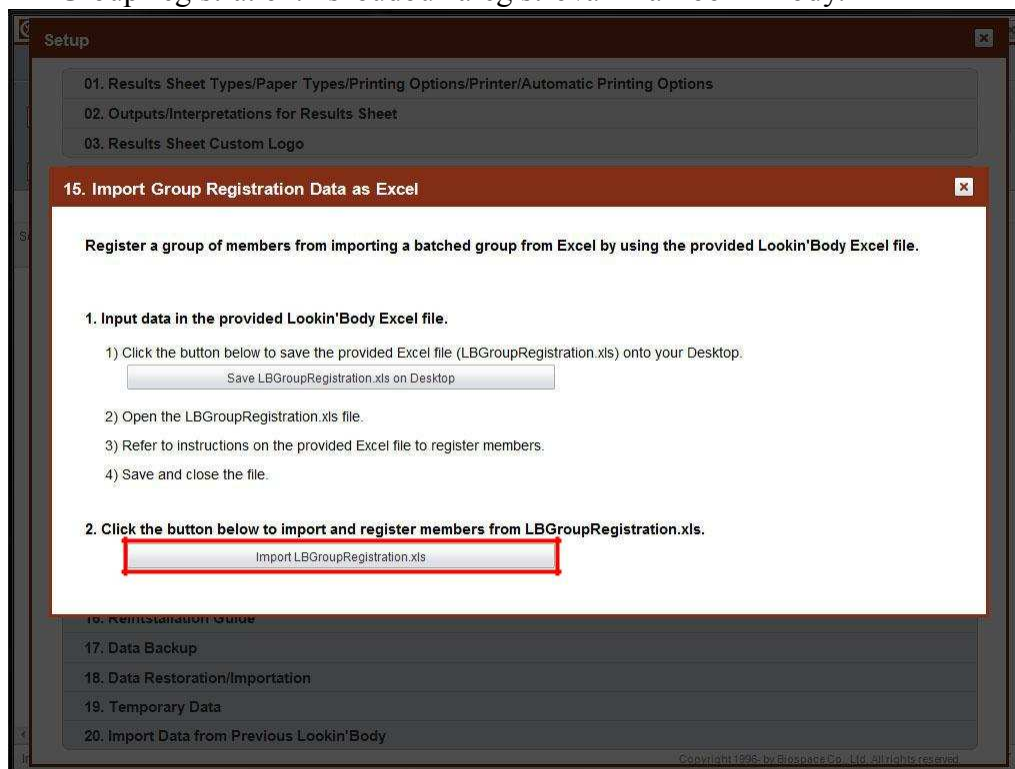
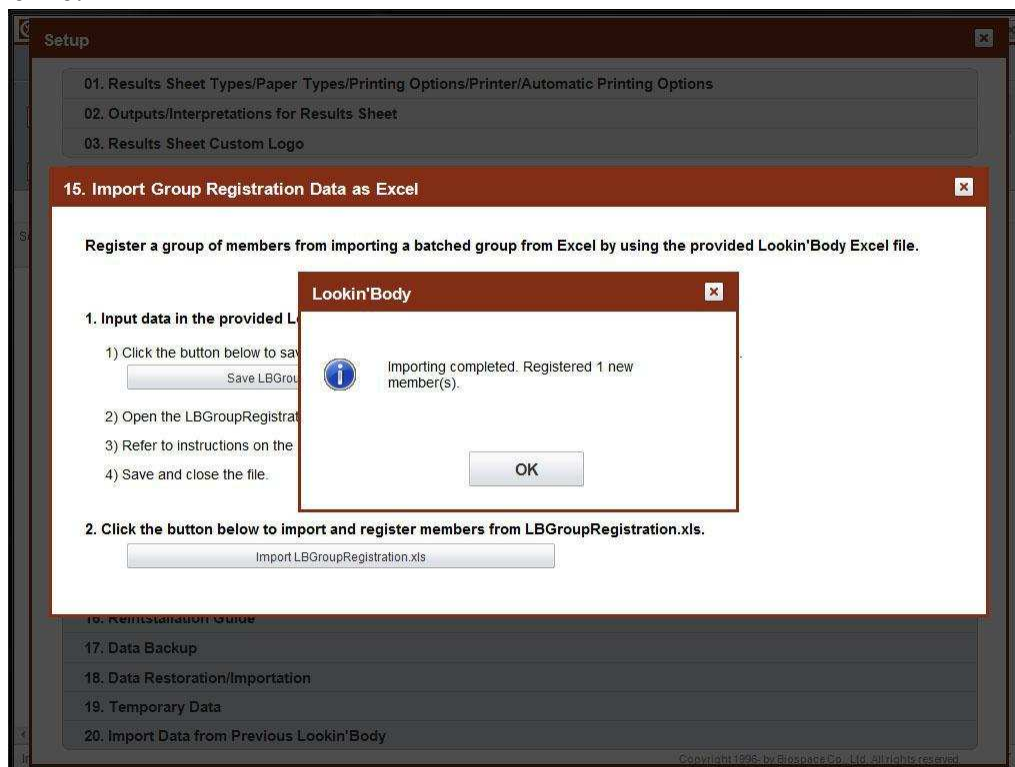
The screenshot shows the Lookin'Body120 software interface. It is divided into several sections:

- Select Member:** Contains search fields for Name or ID and InBody Test Date (09.27.2013 - 09.27.2013), along with buttons for Search, List All, and Register New.
- Select Test:** Features a large 'InBody Test' button.
- Manage Results:** Includes buttons for Print, E-mail, and Edit.
- Searched Member(s) (2 person(s)):** A table listing members with columns for Name, ID, Height, Age, Gender, Member Info, and Health Report. The table shows two members: Jane Doe and John Doe. John Doe is selected with a checked checkbox.
- User's Guide:** Provides instructions: 'First, select a member. 'Select Test' or 'Manage Results' on top after selecting.'

Select	Name	ID	Height (ft.in.)	Age	Gender	Member Info.	Health Report
<input type="checkbox"/>	Jane Doe	130927-2	6ft. 06.0in.	26	Female	Member Info...	InBody
<input checked="" type="checkbox"/>	John Doe	130927-1	6ft. 06.0in.	26	Male	Member Info...	InBody

At the bottom of the window, it shows 'InBody570 : Connected (USB)' and 'Ver.LB120.1.0.0.7'.

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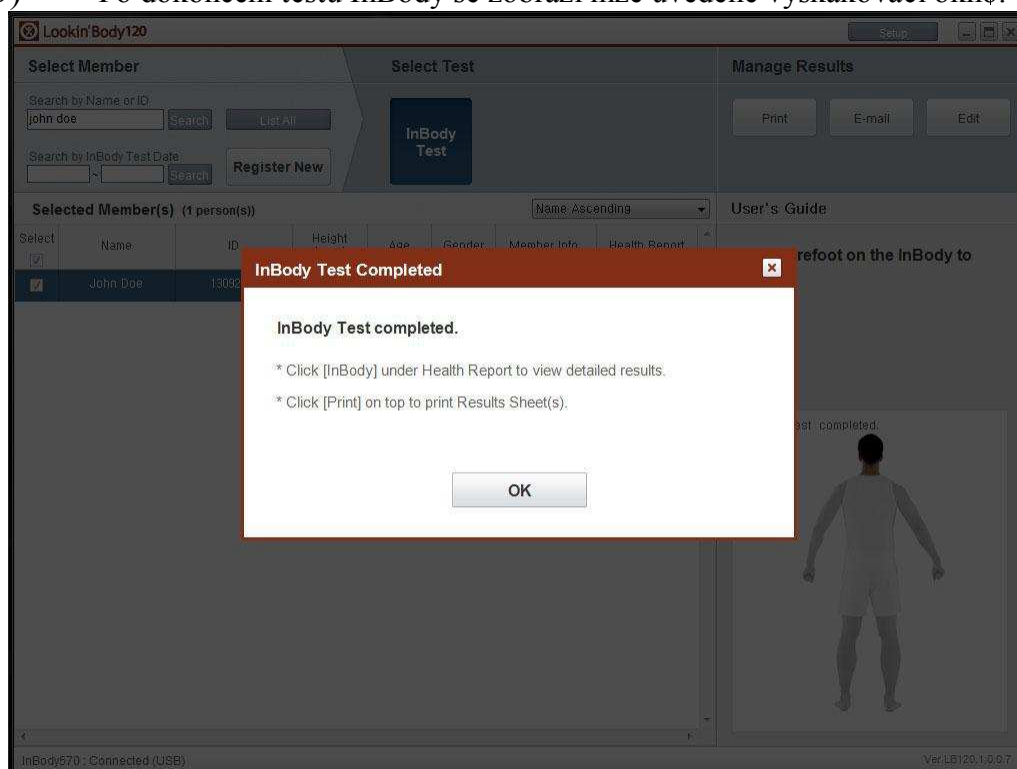
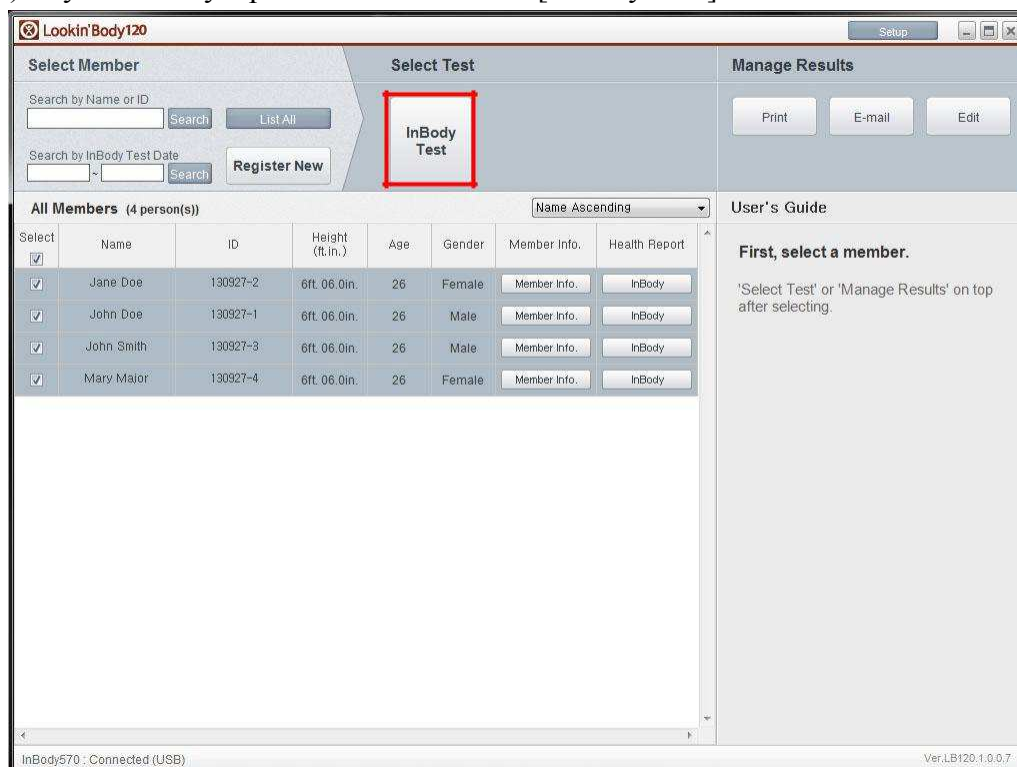
Settings

### Save Test Results

Test results sent from the InBody are listed below.  
Select test results to save in Lookin'Body. Unselected test results will not be saved.

Test results without ID, height, date of birth (or age), and gender will be saved in [Settings]→[19. Temporary Data].

\* If the InBody Test were taken without inputting age and gender, the test results cannot be saved in Lookin'Body even if you input age and gender below. Only member information will be saved.

Select	Name	ID	Height (cm)	Date of Birth or Age			Gender	Test Date / Time	
				Date of Birth					Age
				Month	Day	Year			
<input checked="" type="checkbox"/>		123456789	168.00				26.0	Female	08.05.2013, 11:46

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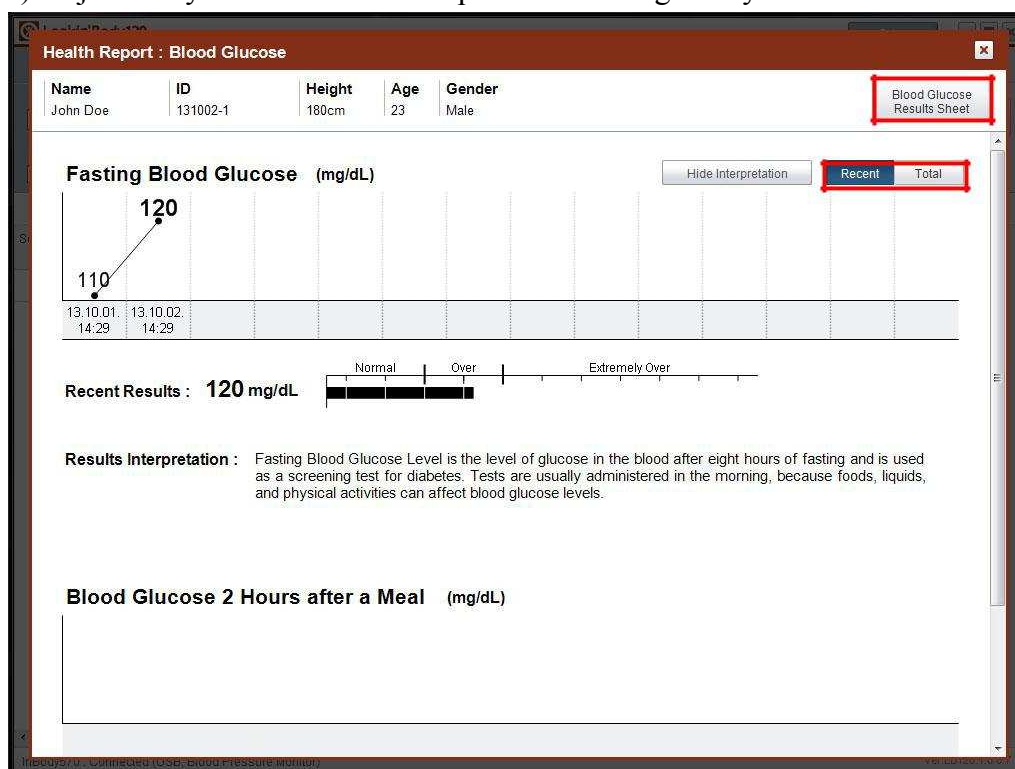
The screenshot shows the Lookin'Body120 software interface. The window title is "Lookin'Body120". The interface is divided into several sections:

- Select Member:** Contains search fields for "Name or ID" (with "john doe" entered) and "InBody Test Date". Buttons for "Search", "List All", and "Register New" are present.
- Select Test:** Contains buttons for "InBody Test" and "Blood Pressure Test".
- Manage Results:** Contains buttons for "Print", "Email", and "Edit".
- Searched Member(s) (1 person(s)):** A table with columns: Select, Name, ID, Member Info., and Health Report. The "Health Report" column contains buttons for "InBody", "Blood Pressure", and "Blood Glucose". The "Blood Glucose" button is highlighted with a red box.
- User's Guide:** Contains the text: "First, select a member. 'Select Test' or 'Manage Results' on top after selecting."

At the bottom of the window, the status bar shows "InBody570 : Connected (USB, Blood Pressure Monitor)" and "Ver:LB120:1.0.0.7".

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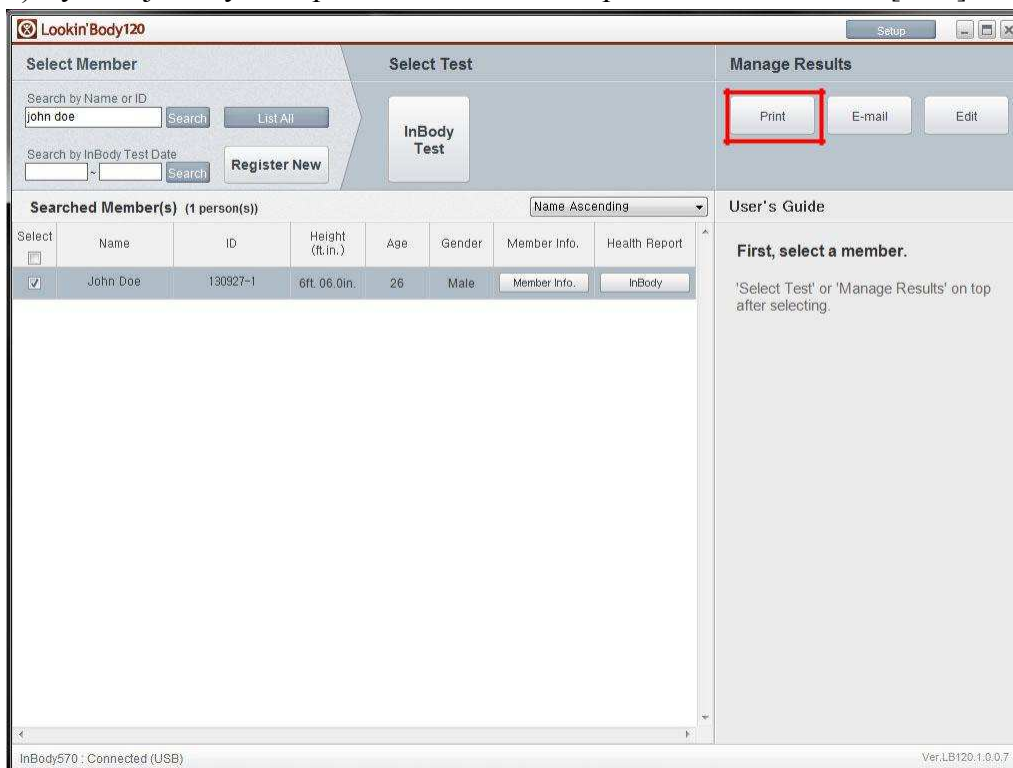
The screenshot shows the Lookin'Body120 software interface. The window title is "Lookin'Body120". The interface is divided into several sections:

- Select Member:** Contains search fields for "Name or ID" (with "john doe" entered) and "InBody Test Date". Buttons for "Search", "List All", and "Register New" are present.
- Select Test:** Features a large "InBody Test" button.
- Manage Results:** Contains three buttons: "Print" (highlighted with a red box), "E-mail", and "Edit".
- Searched Member(s) (1 person(s)):** A table with columns: Select, Name, ID, Height (ft.in.), Age, Gender, Member Info., and Health Report. The table contains one entry for "John Doe" with ID "130927-1", Height "6ft. 06.0in.", Age "26", and Gender "Male".
- User's Guide:** Contains the text "First, select a member." and a note: "'Select Test' or 'Manage Results' on top after selecting."

At the bottom of the window, the status bar shows "InBody570 : Connected (USB)" on the left and "Ver.LB120.1.0.0.7" on the right.

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The screenshot shows the Lookin'Body120 software interface. The window title is "Lookin'Body120" and it includes a "Setup" button in the top right corner. The interface is divided into three main sections: "Select Member", "Select Test", and "Manage Results".

**Select Member:** Contains search fields for "Name or ID" (with "john doe" entered) and "InBody Test Date". Buttons include "Search", "List All", and "Register New".

**Select Test:** Features a button labeled "InBody Test".

**Manage Results:** Contains buttons for "Print", "E-mail", and "Edit". The "Start Print" button is highlighted with a red rectangle.

**Selected Member(s) (1 person(s)):** A table with columns: Select, Name, ID, Height (ft.in.), Age, Gender, Member Info., and Health Report. The table contains one entry for "John Doe" with ID "130927-1", Height "6ft. 06.0in.", Age "26", and Gender "Male".

**User's Guide:** Contains instructions: "Click [Start Print].", "Results : InBody Results Sheet Sheet(s)", and "Copies: 1 Copy(s)". It also includes three asterisked notes: "\* Results Sheet types and number of copies can be changed in Setup.", "\* The most recent results will be printed.", and "\* If there are no test results, Results Sheet(s) will not print."

At the bottom left, it says "InBody570 : Connected (USB)" and at the bottom right, "Ver.LB120.1.0.0.7".

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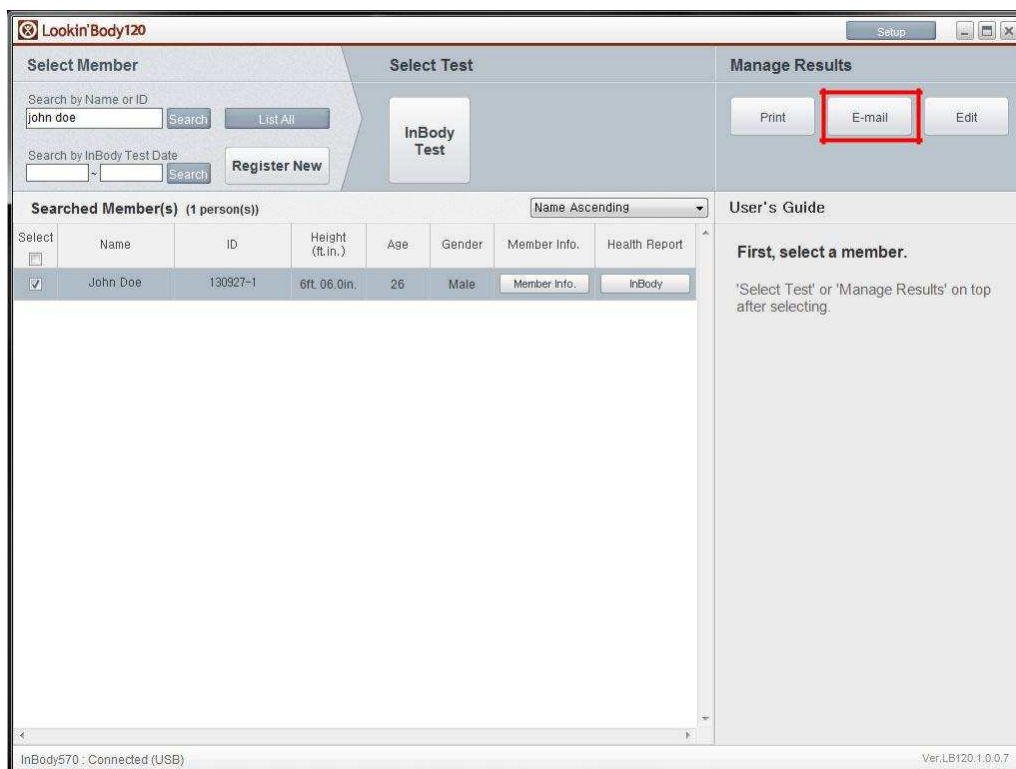
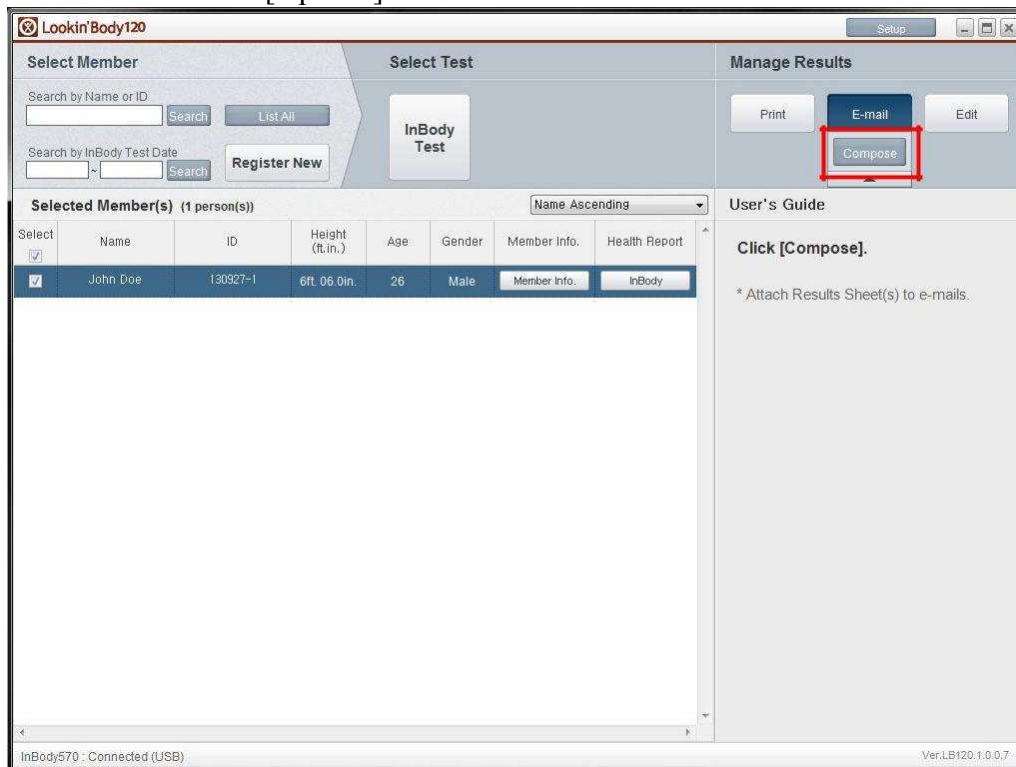
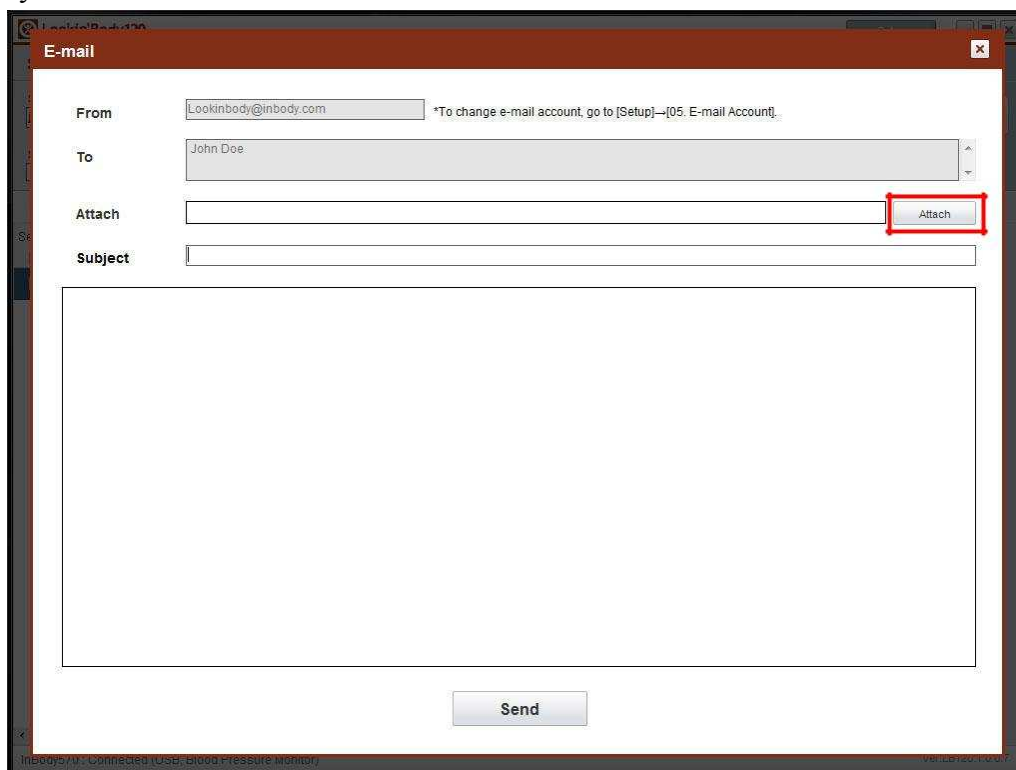
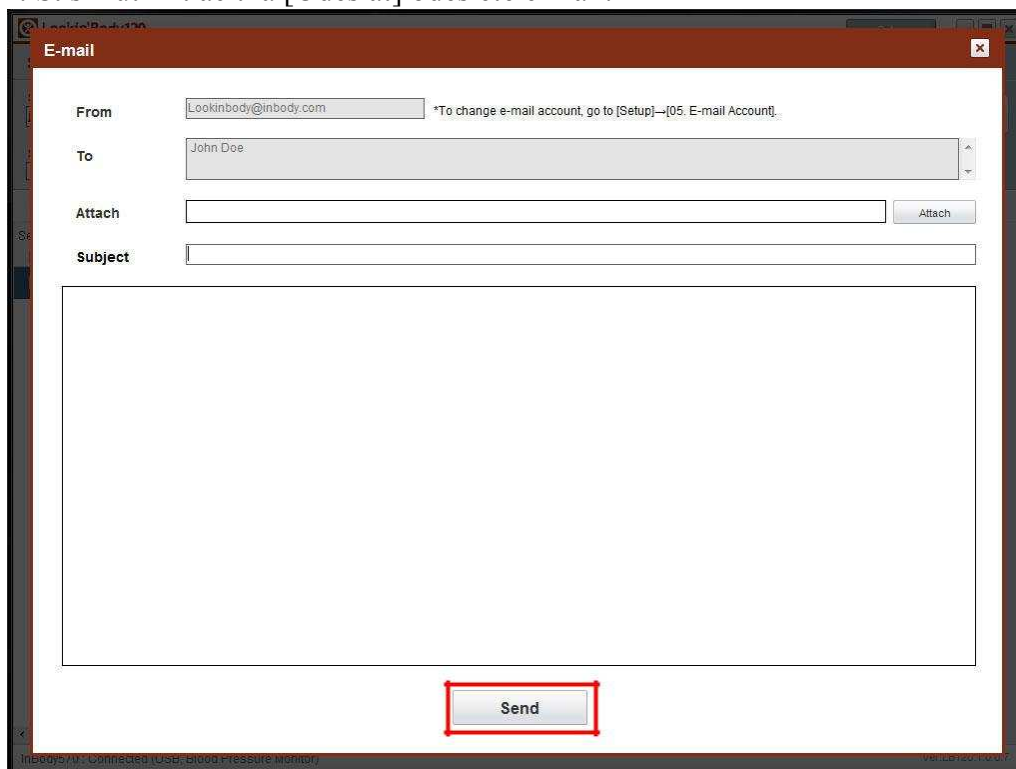
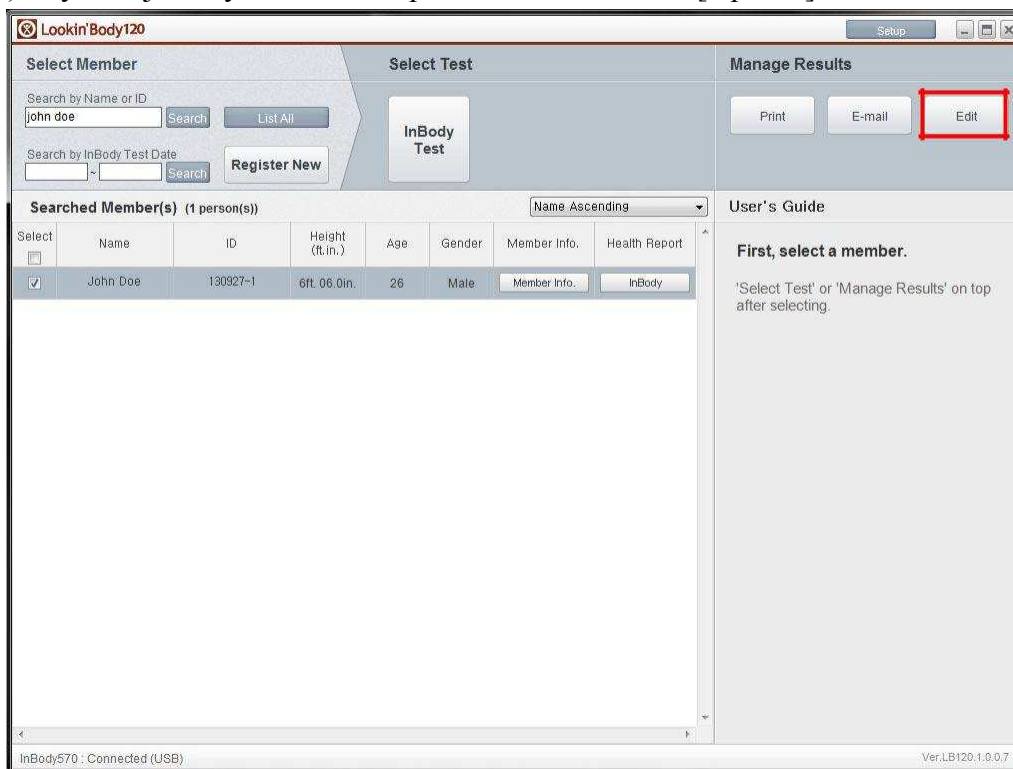
- **Register New :** Click an empty cell below and input all required Member Info. (Required) to register a new member.
- **Edit Member Info. :** Click cell to edit data.
- **Delete Member(s) :** Select members then click [Delete] on the top right corner to permanently delete.

(Deleting a member will permanently erase all results. Results cannot be restored)

Select	Member Info. (Required)							Member Info. (Optional)		
	Name	ID	Height	Gender	Date of Birth			Age	Mobile Number	Telephc
					Month	Day	Year			
<input type="checkbox"/>	John Doe	130927-1	6ft. 06.0in.	Male	03	10	1987	26		
<input type="checkbox"/>										
<input type="checkbox"/>										
<input type="checkbox"/>										
<input type="checkbox"/>										
<input type="checkbox"/>										
<input type="checkbox"/>										

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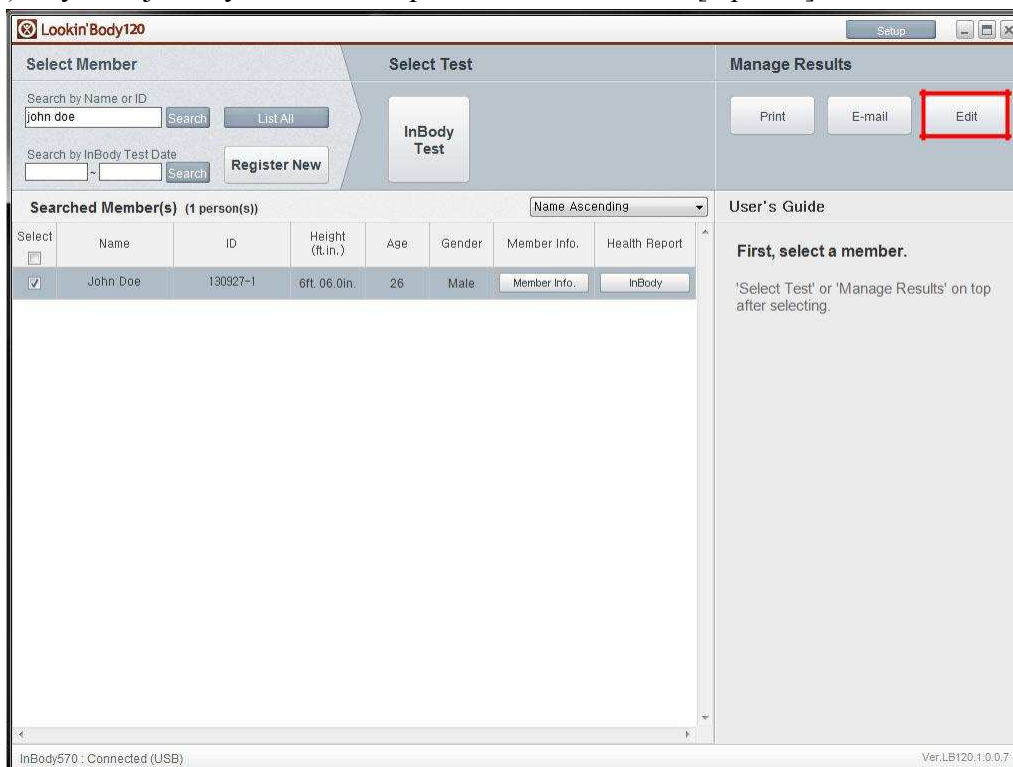
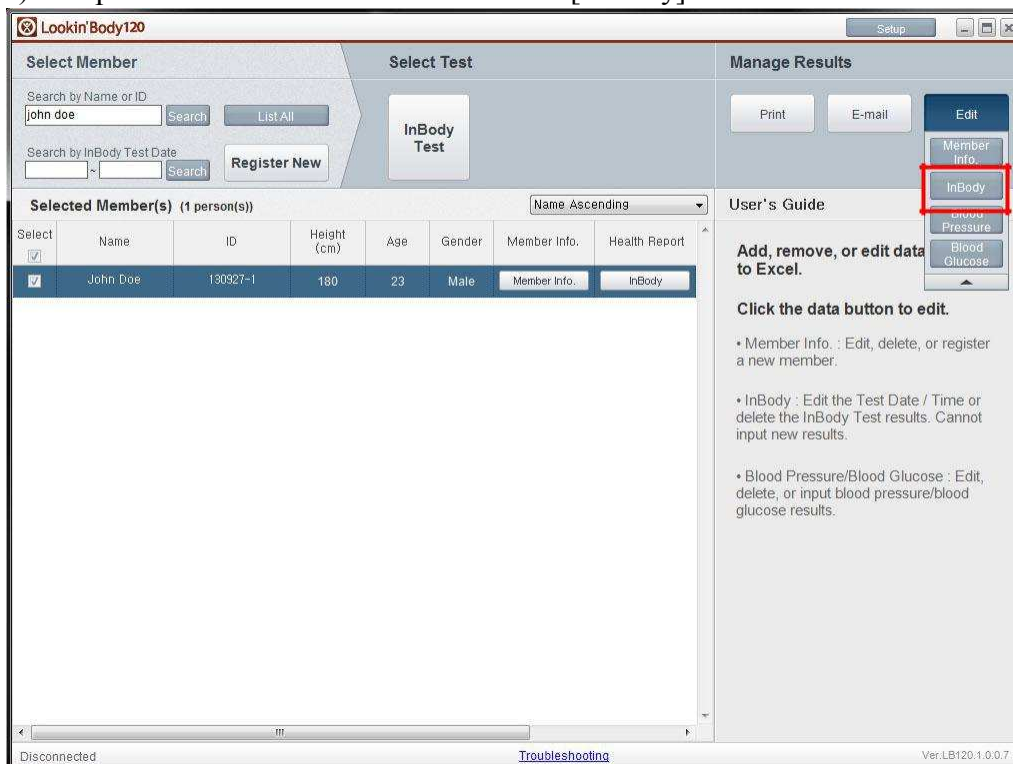
• **Edit Test Date / Time:** Click cell to edit saved data.

• **Delete Results:** Click checkbox on left to select results, then click [Delete] on top right corner to permanently delete.

Select	Name	ID	Test Date / Time	Weight	TBW (Total Body Water)	ICW (Intracellular Water)	ECW (Extrac
<input type="checkbox"/>	John Doe	130927-1	09.27.2013. 11:15	105.9	76.7	46.7	30
<input type="checkbox"/>	John Doe	130927-1	09.27.2013. 11:00	105.2	76.3	46.3	30
<input type="checkbox"/>	John Doe	130927-1	09.27.2013. 10:50	107.1	77.6	47.2	30
<input type="checkbox"/>	John Doe	130927-1	09.27.2013. 10:44	106.9	77.4	47.2	30

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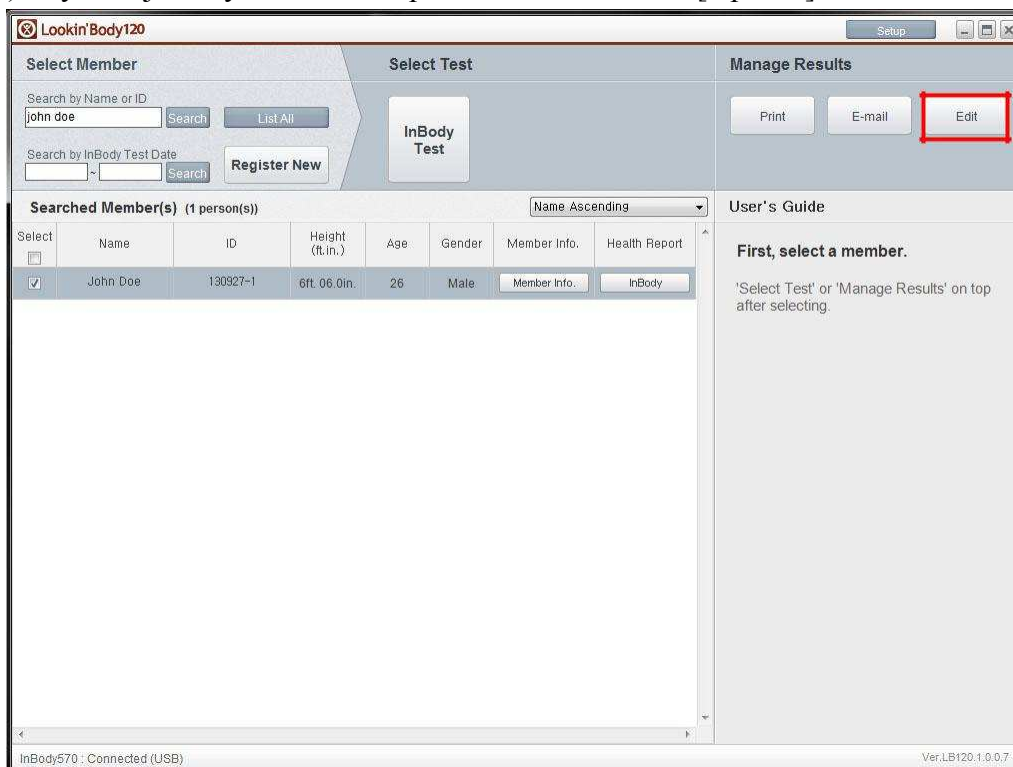
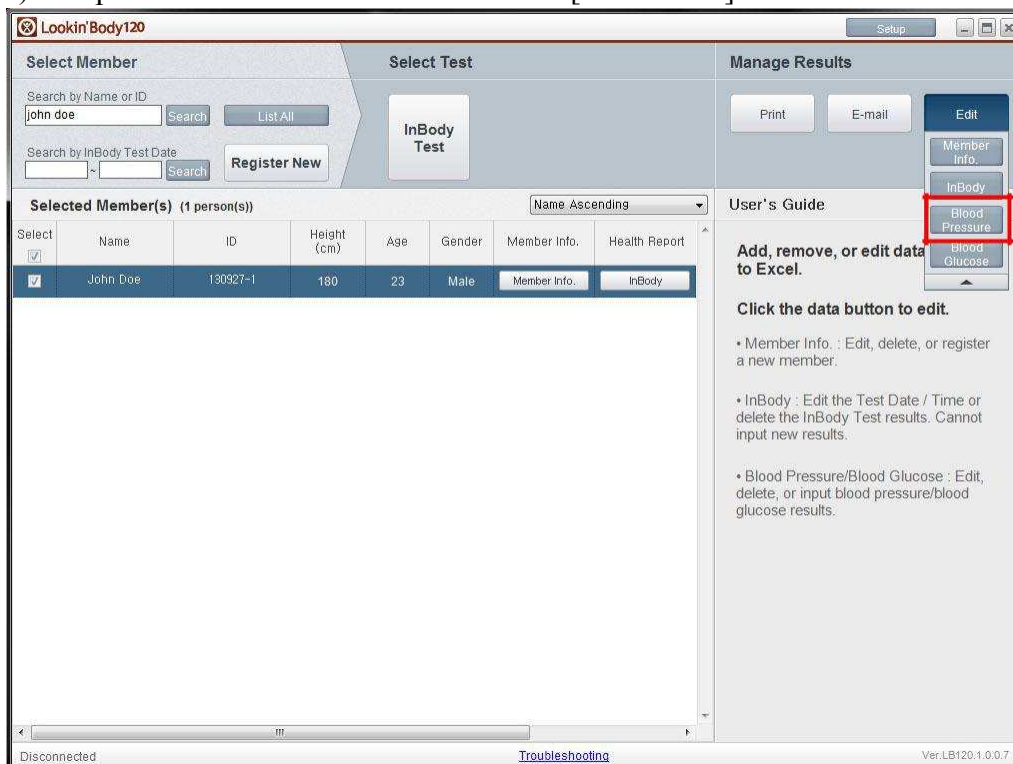
**Edit : Blood Pressure**

Add, remove, or edit data similar to Excel. (Double-click the cell to position cursor.)

- Add Results:** Right-click cells under Name, then click [Add Results] to create a new row right below.
- Input Results:** Click cell to input Systolic, Diastolic, and Pulse Rate. Mean Artery Pressure, Pulse Pressure, and Rate Pressure Product will be calculated automatically.
- Edit Results:** Click cell to edit saved data.
- Delete Results:** Click checkbox on left to select results, then click [Delete] on top right corner to permanently delete.

Select	Name	ID	Blood Pressure Test Date / Time	Sys. (mmHg)	Dia. (mmHg)	Pulse (bpm)	MAP (mmHg)	PP (mmHg)	RPP
<input type="checkbox"/>	John Doe	130927-1	2013.10.02. 18:42	123	73	77	89	50	9471
<input type="checkbox"/>	John Doe	130927-1	2013.10.02. 14:22	125	86	90	99	39	11250

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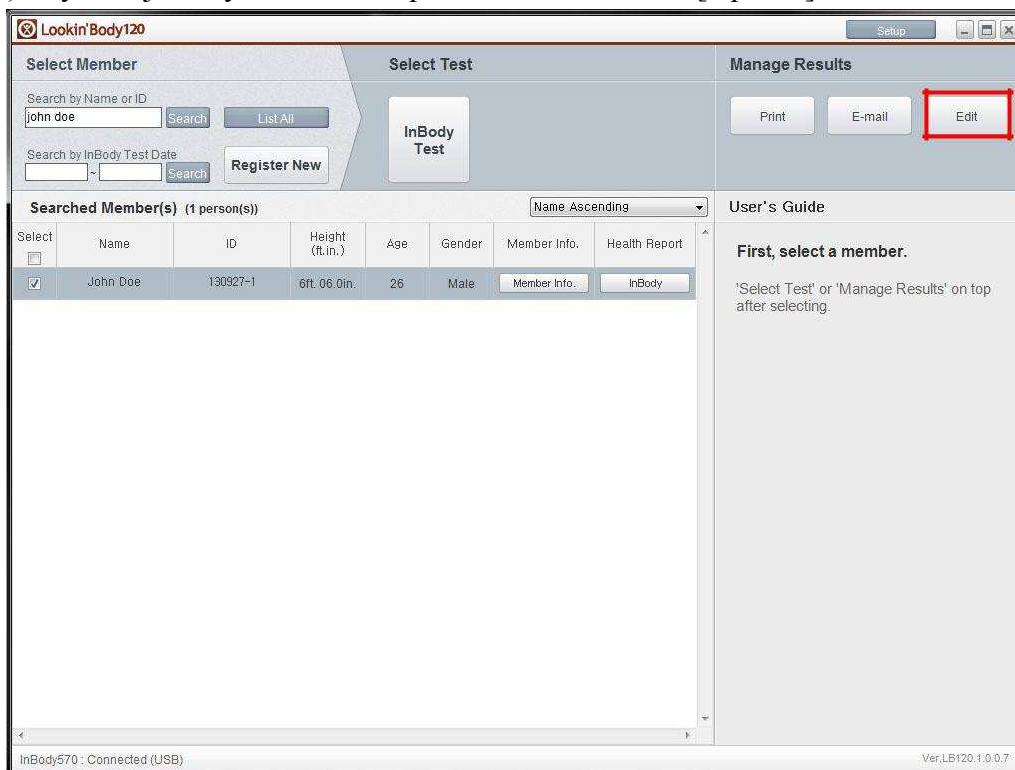
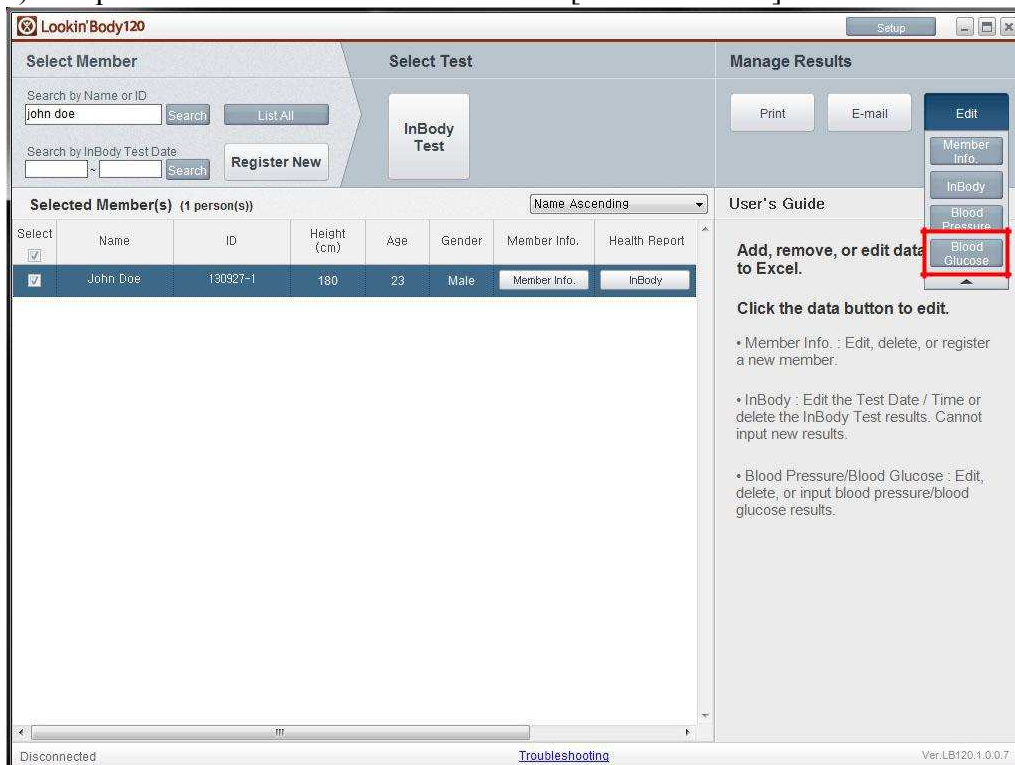
**Edit : Blood Glucose**

Add, remove, or edit data similar to Excel. (Double-click the cell to position cursor.)

- **Add Results:** Right-click cells under Name, then click [Add Results] to create a new row right below.
- **Input Results:** Click cell to input results.
- **Edit Results:** Click cell to edit saved data.
- **Delete Results:** Click checkbox on left to select results, then click [Delete] on top right corner to permanently delete.

Select	Name	ID	Blood Glucose Test Date / Time	Blood Glucose (mg/dL)	Meal Before Test (Time Lapsed)
<input type="checkbox"/>	John Doe	130927-1	2013.10.02.14:29	120	Fasting
<input type="checkbox"/>	John Doe	130927-1	2013.10.01.14:29	110	Fasting

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01. Results Sheet Types/Paper Types/Printing Options/Printer/Automatic Printing Options

Select options below.

1. Select Results Sheet type to print.

**InBody Results Sheet**

This Results Sheet shows InBody Test results with graphs.

**\*When a member under 18 years old is taking the InBody Test, Would you replace the InBody Results Sheet with InBody Results Sheet for a child?**

Yes, replace the InBody Results Sheet with the InBody Results Sheet for a child

No, do not replace the InBody Results Sheet with the InBody Results Sheet for a child

This Results Sheet shows InBody Test results with graphs including Child Growth Curve.

**InBody Results Interpretation**

\*Only available when the InBody Results Sheet is selected.

This Results Interpretation shows the explanation of the InBody Test results.

**Body Composition History Results Sheet**

This Results Sheet tracks body compositional change (Weight, Skeletal Muscle Mass, Body Fat Mass, Percent Body Fat, ECW Ratio, and Leg Lean Mass).

**\*Select the body composition history graph type.**

Recent  
(A cumulative graph that shows the progress over 15 most recent results.)

Total  
(A cumulative graph that shows the progress from first test results to the most recent results.)

**Blood Pressure Results Sheet**

\*If automatic printing option is enabled under "5. Select whether to print Results Sheets automatically after each InBody Test" below, this Results Sheet will be printed automatically after the InBody Test.

This Results Sheet shows blood pressure test results with graphs.

**Blood Glucose Results Sheet**

\*If automatic printing option is enabled under "5. Select whether to print Results Sheets automatically after each InBody Test" below, this Results Sheet will be printed automatically after the InBody Test.

This Results Sheet shows blood glucose test results with graphs.

2. Select the paper type to print.

Blank A4 Paper : Results Sheet(s) will be printed on the blank A4 paper.

InBody Paper : Results Sheet(s) will be printed on the InBody paper. Framework is color-printed.

\*To adjust print alignment, click [Printing Alignment].

3. Select number of copies to print.

1 copy ( )  2 copies ( )

4. Select a printer.

Samsung Universal Print Driver

5. Select whether to print Results Sheets automatically after each InBody Test.

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**02. Outputs/Interpretations for Results Sheet**

Select outputs/interpretations to print on the right side of the Results Sheet. Bracketed values show required space. Check Output is a result of the InBody Test. Interpretation is an explanation of the output.

**InBody** **Biospace**

**InBody Score Output[10]** : This score shows the evaluation of your body composition, which includes muscle, fat, and water in the body.

**Body Type Output[21]** : The body type based on BMI and Percent Body Fat.

**Weight Control Output[10]** : See how your body measures up to the recommended Weight, Muscle Mass, and Body Fat Mass for a good balance. Blank spaces will be available for use if values are not printed.

With Values  Without Values

**Weight Control**

Target Weight	75.0 kg
Weight Control	0.0 kg
Fat Control	0.0 kg
Muscle Control	0.0 kg

**Nutrition Evaluation Output[9]** : Evaluates whether the amount of Protein, Minerals, and Body Fat is adequately distributed in the body. Check marks can be omitted per preference.

With Check Marks  Without Check Marks

**Nutrition Evaluation**

Protein	<input checked="" type="checkbox"/> Normal <input type="checkbox"/> Deficient
Minerals	<input checked="" type="checkbox"/> Normal <input type="checkbox"/> Deficient
Body Fat	<input checked="" type="checkbox"/> Normal <input type="checkbox"/> Deficient <input type="checkbox"/> Excessive

**Obesity Evaluation Output[9]** : Evaluates obesity based on BMI and Percent Body Fat. Check marks can be omitted per preference.

With Check Marks  Without Check Marks

**Obesity Evaluation**

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03. Results Sheet Custom Logo

The top right corner is available for custom logo use.  
Select logo option.

Load logo from the InBody.  Text-only logo  No logo

Load

Preview

**Lookin'Body**

ID	Height	Age	Gender	Date/Time
SAMPLE	180cm	32	Male	2013.04.29 11:33

Body Composition Analysis

	Water	Total Body Water	Lean Body Mass	Weight
Intracellular Water (i)	33.1			
Extracellular Water (e)	19.6	51.7	71.2	

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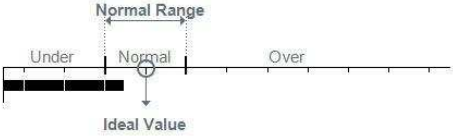
**04. Normal Range**

Set normal ranges below. The graph will be drawn accordingly.

• **Normal Range/Ideal Value**

• **BMI (kg/m<sup>2</sup>)**

– **Normal Range**

**Option 1** (18.5 ~ 25.0) \* Recommended by WHO.

**Option 2** (18.5 ~ 23.0) \* Recommended by WHO.

**Option 3** 18.5 ~  (Min 23 / Max 29)

– **Ideal Value** (Ideal BMI value may shift the ideal weight value.)

**Option 1** Male 22 / Female 21.5

**Option 2** Male 22 / Female 21

• **Percent Body Fat (%)** (Ideal values of percent body fat are 15% for male and 23% for female.)

– **Normal Range**

**Male**  ~  (Min 5 / Max 14) (Min 10 / Max 50)

**Female**  ~  (Min 5 / Max 22) (Min 24 / Max 50)

• **Waist-Hip Ratio** (Ideal WHR value is the normal range median.)

– **Normal Range**

**Male**  ~  (Min 0.5) (Max 1.5)

**Female**  ~  (Min 0.5) (Max 1.5)

**Standard Child Growth Curve**

The Growth Graph on the InBody Results Sheet for Children compares the height and weight among peers of the same age group.

Select standard child growth curve within the Growth Graph to compare with different peers.

Select the standard child growth curve type.

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**Set an account to send e-mails.**

**1. Select account.**

**InBody account:** E-mail using the account provided by BIOSPACE.  
LookinBody@inbody.com is the default e-mail shown to members and cannot be changed.

**User account:** E-mail using your own account. Information below is required.

**2. Input the required information.**

• **Host**   
\* Example : mail.inbody.com

• **User**   
\* Example : LookinBody@inbody.com  
\* This e-mail address will be shown to your members.

• **Name**   
\* Example : InBody  
\* This user name will be shown to your members.

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The screenshot shows a window titled "06. InBody Model/Country/Language/Units/Date Format/Password". It contains the following sections:

- Select the InBody model.** A dropdown menu with "InBody570" selected.
- Select country.** A dropdown menu with "UK" selected.
- Select language.** A dropdown menu with "English" selected.
- Select unit.** Two radio buttons: "kg/cm" (selected) and "lbs/ft.in".
- Select date format.** Three radio buttons: "Year.Month.Day." (selected), "Month.Day.Year.", and "Day.Month.Year."
- Click the button below to set a password.** Two buttons: "Set Password for Setup" and "Set Master Password".

Below the buttons are two lines of explanatory text:

- Next to "Set Password for Setup": "Set a password to prevent unauthorized access to the Setup."
- Next to "Set Master Password": "Set a master password to enhance data security."

At the bottom center is a "Save" button.

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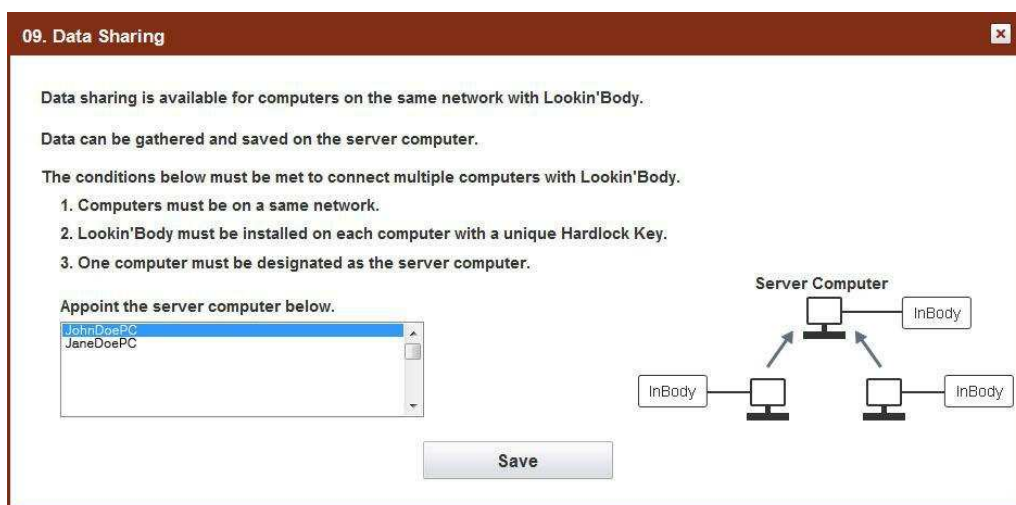
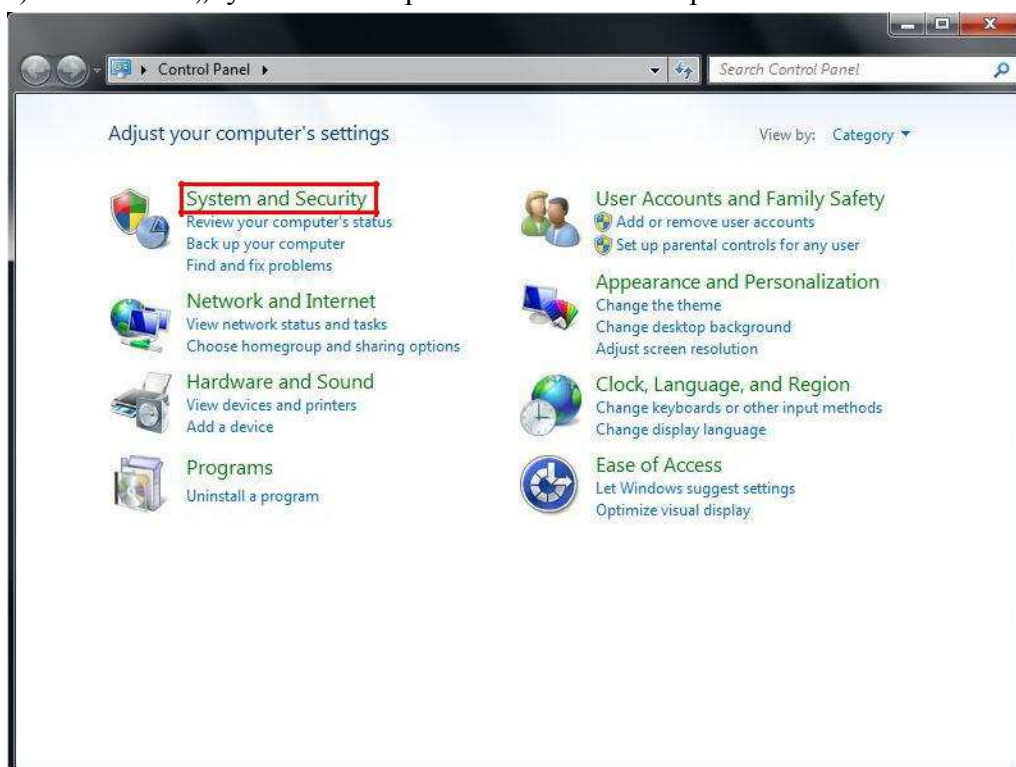
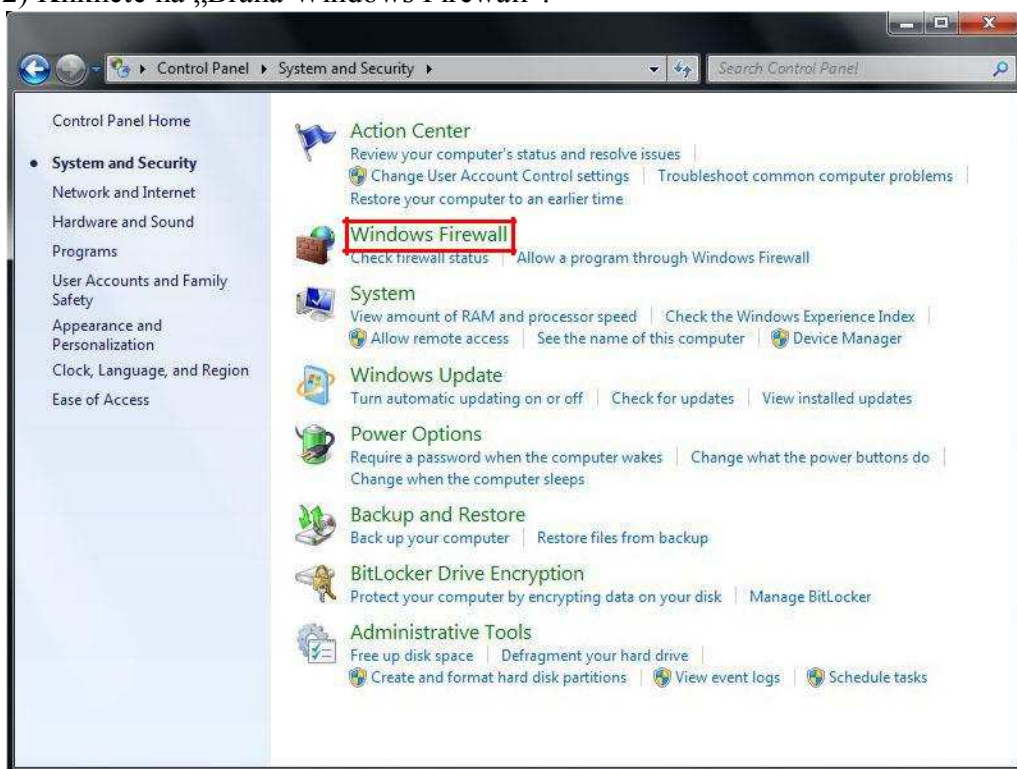
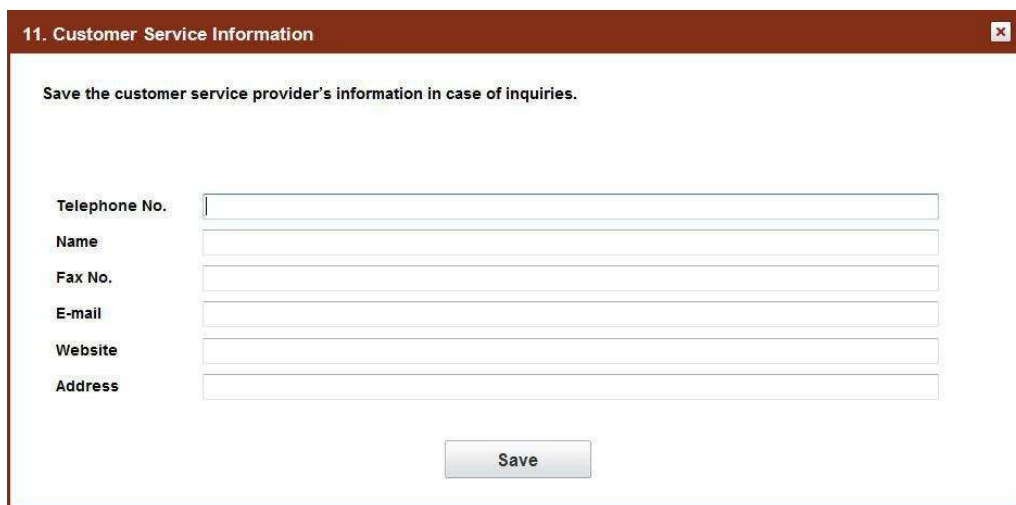
11. Customer Service Information

Save the customer service provider's information in case of inquiries.

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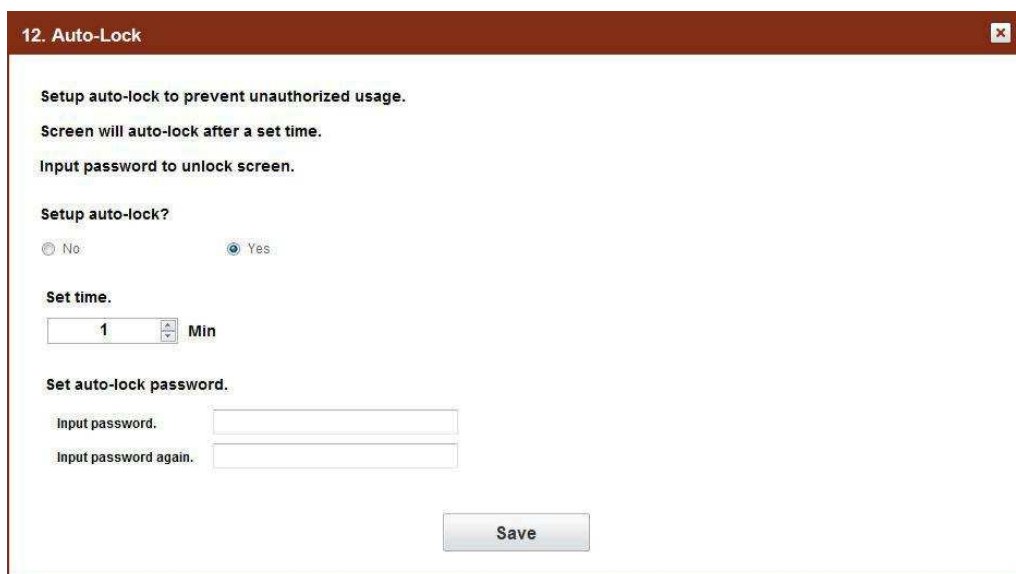
12. Auto-Lock

Setup auto-lock to prevent unauthorized usage.  
Screen will auto-lock after a set time.  
Input password to unlock screen.

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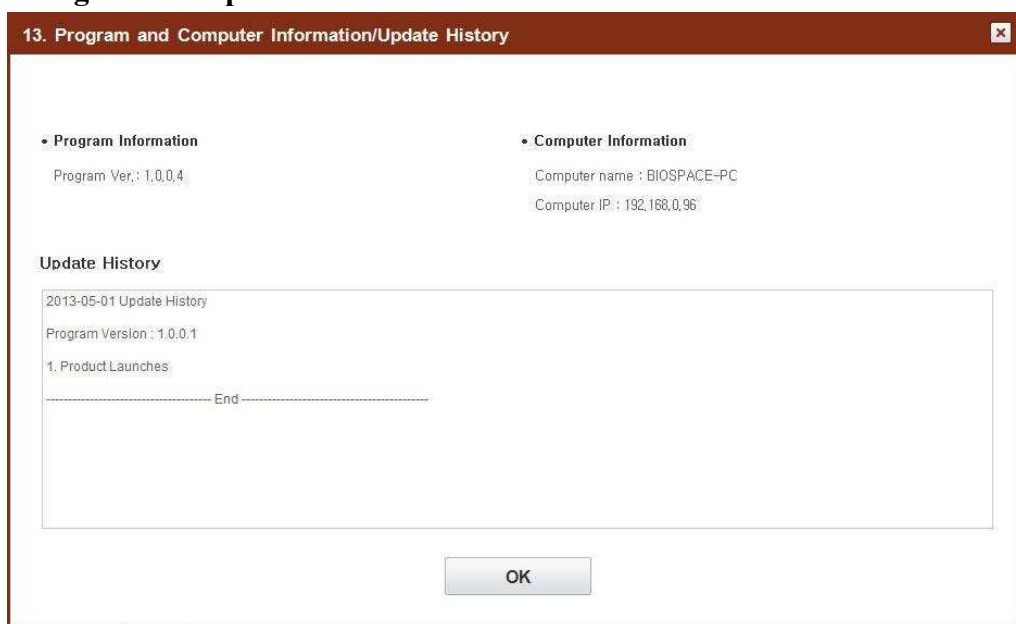
**15. Import Group Registration Data as Excel**

Register a group of members from importing a batched group from Excel by using the provided Lookin'Body Excel file.

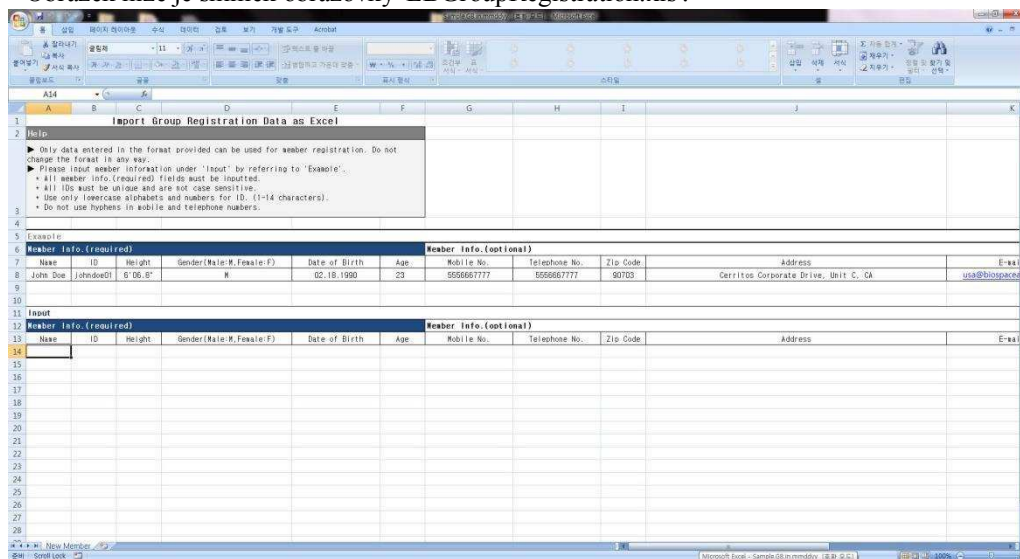
**1. Input data in the provided Lookin'Body Excel file.**

- 1) Click the button below to save the provided Excel file (LBGroupRegistration.xls) onto your Desktop.
- 2) Open the LBGroupRegistration.xls file.
- 3) Refer to instructions on the provided Excel file to register members.
- 4) Save and close the file.

**2. Click the button below to import and register members from LBGroupRegistration.xls.**

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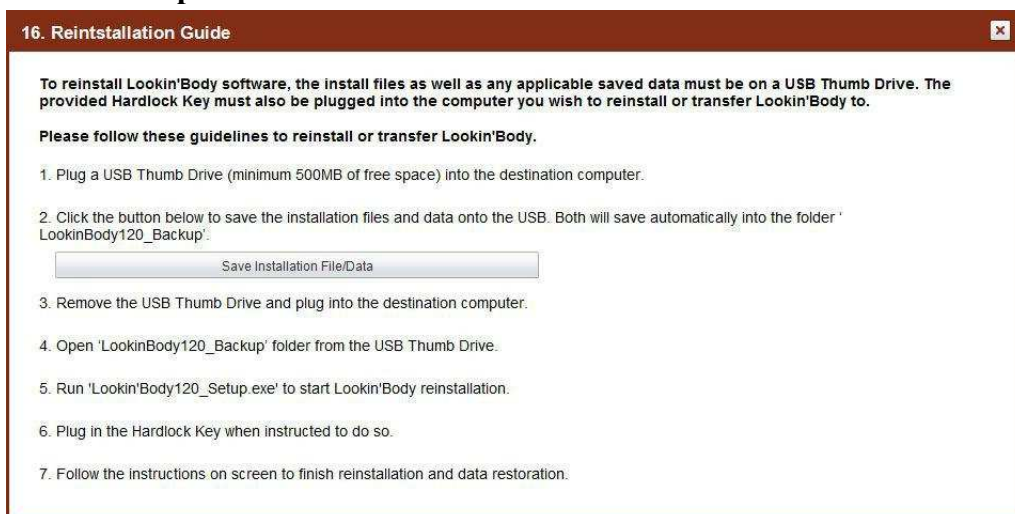
## 18. Obnova/import dat

**18. Data Restoration/Importation**

Data Restoration will restore member information, test results, and user settings from a previous Lookin'Body backup file. Data Importation will import saved data from the InBody by using a USB Thumb Drive.

1. Restore data from a previous Lookin'Body backup file
  - 1) Click [Data Restoration/Importation] below.
  - 2) Select the folder that has the backup file.
2. Import data saved from the InBody by using the USB Thumb Drive
  - 1) Click [Data Restoration/Importation] below.
  - 2) Select the USB Thumb Drive used to export data from the InBody. Click the 'inbody' folder, then select the 'lookinbody' folder.

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## 19. Dočasná data

**19. Temporary Data**

Temporarily saved test results are shown below

To save test results in Lookin'Body

1. Input ID, height, date of birth (or age), and gender.
2. Click the checkbox on the left to select data, then click [Save] at the bottom.

\*If the InBody Test was taken without entering age or gender, the results cannot be saved. Only member information will be saved.

To delete temporary data, select the checkboxes respective to said data, then click the [Delete] located on the top right corner.

Select	Name	ID	Height (ft.in.)	Date of Birth or Age				Gender	Test Date / Time
				Date of Birth			Age		
				Month	Day	Year			
<input type="checkbox"/>									

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©1996 ~ Biospace Co., Ltd. All rights reserved. (BM-ENG-I1-A-131002)